

# Health and Safety Engagement Checklist

## Health Services

### GENERAL

Are there any signs of excessive wear and damage visually?

☐ Yes☐ No☐ Na

Is the base, framework, bed ends, and side rails in good condition?

☐ Yes☐ No☐ Na

Are the rail welds free of damage and cracks?

☐ Yes☐ No☐ Na

Are there any signs of warping, bending, or damage on foot and head sections?

☐ Yes☐ No☐ Na

Are all rivets and bolts tightened securely and functioning properly?

☐ Yes☐ No☐ Na

Are the surfaces and links intact?

☐ Yes☐ No☐ Na

Do casters roll and lock as required?

☐ Yes☐ No☐ Na

Does the bed lower and raises properly?

☐ Yes☐ No☐ Na

Does the foot lower and raise properly?

☐ Yes☐ No☐ Na

Does the head lower and raise properly?

☐ Yes☐ No☐ Na

Are electrical parts free of excessive wear or damage?

☐ Yes☐ No☐ Na

Are power, pendants, and motor cords free of excessive wear, cuts, or signs of chafing?

☐ Yes☐ No☐ Na

Are all plugs damage-free and fully attached?

☐ Yes☐ No☐ Na

Is the junction box cable lock properly positioned and locked?

☐ Yes☐ No☐ Na

Are all other mechanical functions in working condition?

☐ Yes☐ No☐ Na

Is the mattress cover free from rips or cracks?

☐ Yes☐ No☐ Na

Does the emergency hand crank work properly?

Yes

No

Na

Are any harsh cleaning agents used to clean the beds?

Yes

No

Na

Is greasing done periodically to actuators, tubes, pins, etc?

Yes

No

Na

Are there any signs of impedance or leakage?

Yes

No

Na

Are damaged parts replaced/repaired as required?

Yes

No

Na

Are records maintained for cleaning and maintenance?

Yes

No

Na

Comments

Image

UPLOAD

Image

UPLOAD

Image

UPLOAD

Inspectors Name/signature:

SIGNATURE

Inspection Date:

DATE

2017-01-01